


DISKOM

PERCUTANEOUS DISCECTOMY PROBE

“THIS ONE WORKS”

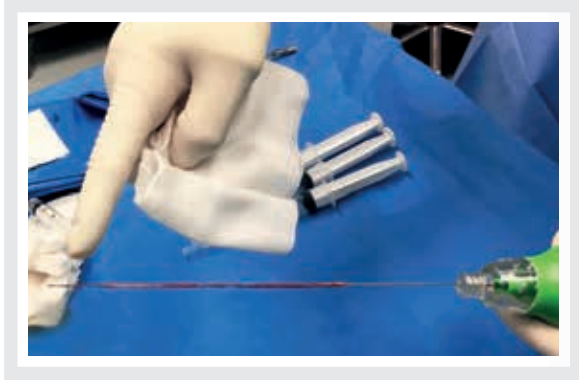

UP TO 2cc COLLECTED



U.S. ONLY

With Diskom pressure on both spinal cord and nerve roots can be significantly reduced by removing disc herniation through a minimally invasive procedure.

DISKOM BENEFITS:

- Preserves discal annulus integrity
- Rapid rehabilitation
- Procedure time: 10 to 15 minutes
- Up to 2cc of disc material collected
- Disc material collected could be used for bioptic procedures
- Available for thoraco-lumbar and cervical spine

SURGICAL TECHNIQUE:

- Introduce the access needle under fluoroscopy into the disc.
- Remove the stylet and introduce the probe into the cannula.
- Lock the probe to the access needle through the luer-lock connection
- Switch the probe on.
- Carry out a continuum movement in antero-posterior direction, for 2/3 minutes. Alternatively, proceed with a rotary movement.
- Disc material is then removed and collected along the probe stylet or into the collection chamber.
- After switching off remove the device.

Diskom permette di ridurre significativamente la pressione sul midollo spinale e sulle radici nervose, rimuovendo l'ernia discale attraverso una procedura mini invasiva.

BENEFIT DISKOM:

- Preserva l'integrità dell'annulus discale
- Permette una rapida riabilitazione
- Durata della procedura: da 10 a 15 minuti
- Raccoglie fino a 2cc di materiale discale
- Il campione raccolto può essere utilizzato per esami bioptici
- Diskom è compatibile con procedure toraco-lombari e cervicali

TECNICA CHIRURGICA:

- Introdurre l'ago di accesso all'interno del disco sotto guida fluoroscopica
- Rimuovere lo stiletto ed inserire il dispositivo nella cannula dell'ago d'accesso
- Collegare il dispositivo all'ago d'accesso attraverso la connessione Luer-Lock
- Mettere in funzione il dispositivo
- Effettuare un movimento continuo in direzione antero-posteriore per 2/3 minuti. In alternativa eseguire un movimento rotatorio
- Il materiale discale viene quindi rimosso e raccolto lungo lo stiletto del dispositivo o all'interno della camera di raccolta
- Rimuovere il dispositivo dopo averlo disattivato

AVAILABLE SIZES

Procedure	Sizes	Length
Cervical	19G	8 cm
Thoracic/Lumbar	17G	16 cm

